INOVO
SENSE

MESO Cell Plus

INSTRUCTIONS FOR USE

DESCRIPTION

• **INOVOSENSE MSO Cell Plus** is a sterile, non-pyrogenic, low-viscosity, colorless, and transparent compound that contains polydeoxyribonucleotide (PDRN). This innovative product is designed for dermal rejuvenation and is composed of salmon DNA fragments. **INOVOSENSE MESO Cell Plus** contains 0.15% lidocaine hydrochloride.

• Each box includes one 2 ml pre-filled disposable glass syringe and two traceable labels.

COMPOSITION

- Polydeoxyribonucleotide: 40 mg

- Lidocaine hydrochloride: 0.15 %

- Phosphate Buffer pH 7 q.s.: 2 ml

INDICATIONS

• **INOVOSENSE MESO Cell Plus** is an injectable biodegradable compound that increases the ability of skin fibroblasts to synthesize collagen and elastin, improving skin well-being. It also has tissue reconstruction, brightening and anti-inflammatory properties.

• **INOVOSENSE MESO Cell Plus** enhances skin firmness and elasticity, reduces wrinkles, and brightens the face, neck, décolletage, and hands. The presence of lidocaine aims to reduce pain during injection process.

USAGE PROCEDURES

• Ensure the injection area is clean and sterile.

• Use a thin gauge needle (31 ~ 32).

• For injection, press the plunger rod carefully until a

small droplet is visible at the tip of the needle.

- Insert the needle at an approximate angle of 30° parallel to the skin surface (superficial or medium dermis).
- The injection should be administered by applying consistent pressure to the plunger rod.
- INOVOSENSE MESO Cell Plus** can be injected using the micro-papular or nappage techniques.
- Recommended treatment frequency: At least three sessions, with a four-week interval, depending on the patient's needs and physician's assessment.
- INOVOSENSE MESO Cell Plus** can be used concomitantly treatments, such as peeling, laser-therapy and dermabrasion.

CONTRAINDICATIONS

- Patients with a history of un-absorbable filler or recent other filler treatment.
- Patients taking NSAIDs, antiplatelet agents, anticoagulants, and immunosuppressants.
- Patients with autoimmune diseases, granulomatous pathology related to sarcoidosis, Osler's endocarditis.
- Use with caution in patients with a history of allergies to fish-derived products.
- Do not use in patients with known sensitivity to lidocaine.
- Do not inject into skin areas presenting active inflammation or infection.
- This product is not recommended for patients with epilepsy or malignant tumors.
- Not recommended for use during pregnancy,

PRECAUTIONS

- Patient must avoid applying makeup for at least 12 hours post-injection, as well as avoid sun exposure, UV rays, cold and heat (e.g., sauna, steam room) for two weeks.
- This product should only be prescribed and administered by a trained physician using appropriate techniques.
- Patients with a history of herpetic rashes may experience recurrent herpetic dermatitis following injection.
- Safety and efficacy have not been established for patients with a sensitivity to keloid formation, hyperpigmentation, or hypertrophic scarring.
- Avoid intravenous injection to prevent infarction (tissue necrosis)
- INOVOSENSE MESO Cell Plus** can be injected with cross-linked hyaluronic acid, but it's important to inject each product at different planes. Granuloma can rarely occur after injecting low-quality cross-linked HA filler in the same plane .

WARNINGS

- Do not use if the packaging is opened or damaged.
- The product is for single use only; do not re-sterilize or reuse.





ADVERSE EFFECTS

- The patient should be fully evaluated by a physician about mid-term and long-term adverse events that may occur.

- If the injection site is pressed, inflammation with itching or pain may occur, which can last up to a week.
- The formation of nodules, sclerosis or discoloration at the injection site may occur.
- Patients should consult a physician immediately if inflammation persists for more than one week or if any unusual adverse events occur.
- The physician should report any unusual adverse events to the manufacturer or supplier.

NEEDLE ASSEMBLY METHOD FOR SYRINGE

- For optimal use of **INOVOSENSE MESO Cell Plus**, it is important that the needle is properly connected to the syringe. See diagrams 1, 2, 3 and 4.
- 1.Firmly hold the glass cylinder of the syringe and the Luer-lock adaptor between the thumb and the index fingers.
 - 2.Grasp the protective cap with the other hand and break the seal.
 - 3.Push & Twist the needle on the syringe until a resistance is felt. Do not over-tight. Overtightening of the needle may lead to the Luer-lock moving and dislodging from the syringe.
 - 4.Keep holding the Luer-lock and remove the sheath from the needle.

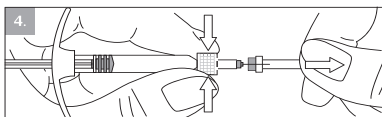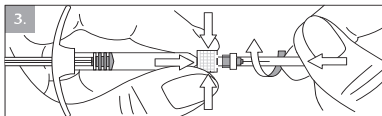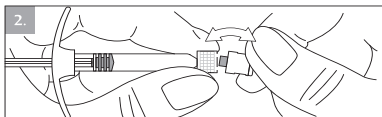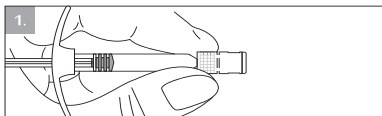STORAGE

•Store between 2-25°C. Protected from light and freezing.

Marketing Authorization Holder:

MEDISA ARA GOSTAR IRANIAN

Number 24, Jooybar Alley, Shad St,

Molla Sadra St, Tehran, Iran

Manufacturer:

ARYA TAJHIZ IBN SINA

Number 21, 20 St, Kaveh Industrial City, Saveh, Iran

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